Microdermoabrasión

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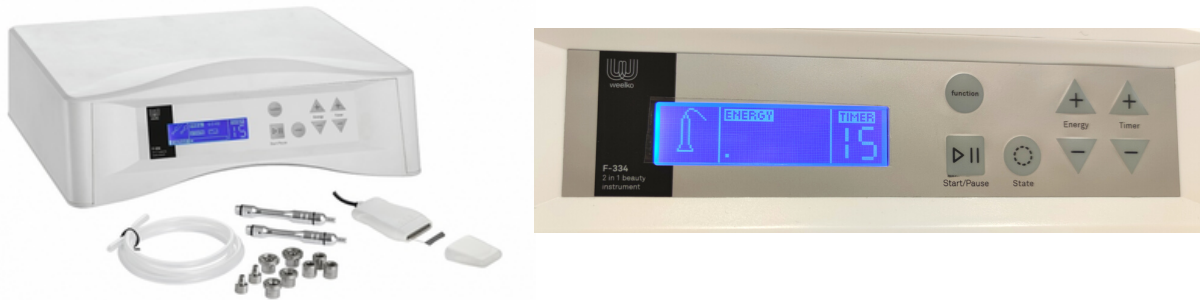
Microdermoabrasion

Non-invasive procedure that is performed at the level of the corneal layer of the skin and seeks the renewal of the epidermis through mechanical exfoliation, helping to remove dead skin by sanding and sweeping, producing cell regeneration, increasing collagen production, and helping to improve the texture and colour of the skin making it smoother and softer.

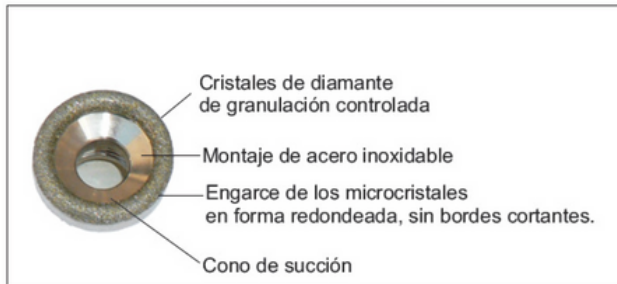
It can be performed on the face, neck, décolleté, arms, hands and back, in order to encourage the formation of new cells to replace those removed. At the end of the microdermabrasion you may feel a burning sensation and dryness of the skin in the area that was exposed to the procedure, this is normal, you can use the cool hammer to reduce the redness or a refreshing mask. However, depending on the intensity of the treatment, the skin may become redder and this should disappear within a few hours.

To activate the microdermabrasion, select Function Diamond dermabrasion and set the suction intensity under energy.

After switching on the equipment, the suction intensity will be regulated and the appropriate diamond tip will be used according to the skin type.

The skin should be clean and dry, and sun exposure should be avoided for at least 1 week beforehand. Refrain from using products with retinoids or glycolic acid in the 10 days before and after.

Punta de Diamante

Microdermabrasion

How to use:

- Connect the tube to the machine and to the microdermabrasion handpiece.
- Place a foam filter before threading the diamond tip. Remove the filter after each treatment.
- Select the tip you wish to use, and then screw it onto the tip of the handpiece, until the rubber O-ring is tightened.
- Adjust the suction intensity with the vacuum regulator according to the table below, keeping in communication with the client for safety.
- Bring the handpiece with the appropriate diamond tip into contact with the skin. Proceed with a slow and smooth gliding motion. Do not leave the handpiece unmoved for too long on the same skin surface, as this may cause injury.
- Gently stretch the skin with one hand while gliding the diamond tip 2 or 3 times over the same area. Respect this point to avoid redness or injury to the skin.
- After the microdermabrasion treatment, cleanse the skin of impurities and dead skin with a soft brush, damp sponge or damp towel.
- Use a brush or cotton swab lightly moistened with alcohol or a disinfectant to clean and disinfect the diamond tip. It is also recommended to sterilise the tips used after each treatment in an ultraviolet steriliser or at high temperature.

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Precautions:

- Avoid use on irritated or injured skin areas.
- When decreased suction is noticed, replace the filter.
- Do not use coarse-grained tips on delicate areas.
- Clean the tips used on each person.
- Avoid bumping and dropping the tips, as this can cause damage and loss of polishing power.
- The microdermabrasion treatment cannot be applied on sensitive areas such as the eyelids, and the eyes must be closed during the treatment, it is advisable to cover them with wet cotton pads.
- It is contraindicated in the following cases: skin tumours, moles, birthmarks, angiomas, eyes, herpes and warts.
- Do not attempt to repair the equipment. In case of failure, please contact your distributor.

F334

Ultrasonic peeling

Treatment designed to deeply cleanse the skin, eliminating dead cells and the horny layer; using a spatula that generates ultrasonic waves and produces vibrations that in contact with the tissue help us to drag all the residues, as well as favouring the stimulation of the skin.

Scrubber (Ultrasonic Peeling) In state we select the type of wave (continuous, pulsed and combined), and in energy we adjust the intensity, in time we choose the treatment time. The spatula should be inclined at a 45° angle, and it is recommended to work with a tonic, if the spatula loses contact with the skin it will stop working automatically. The treatment time is approximately 10-15 minutes. Choose the type of wave in STATE:

Continuous (~): this is a wave that is not interrupted and is used for large surfaces, as in this way the wave emission the manipulator keeps emitting continuously, which generates greater intensity of the treatment, used for thicker skins, with a greater amount of impurities.

Pulsed (ΠΠ): used in small areas, with less thickness and little accumulation of dirt as the wave emission is lower, therefore, the effect will be less powerful and consequently the work will be less. Also used for areas where bone tissue is more prominent.

Combined (Π): is an intermediate point between the two wave emissions.

Make gentle, upward movements.

How to use the Scrubber (Ultrasonic Peeling)

- Connect the equipment correctly.
- Switch it on, and then the machine will be in 'stand by' mode, ready for use.
- Select the desired wave type, 'power' and 'time' level. Please note (the standard default settings are "continuous wave", "maximum power output" and "15 minute timer"), however, you can change any of these settings during the course of the treatment phase.
- Press the "START / PAUSE" button, hold the handpiece between thumb and forefinger respectively. The circuit will close once the wristband is on the client and the spatula is in contact with the skin. The display will also show that the handpiece is active. Hold the spatula at a 45° downward angle and move it only in an upward direction to cleanse the skin, some tonic or gel should be used to work with the ultrasonic peel.
- Once you have finished, remove the handpiece from the skin and the wristband.
- You can also use the back of the spatula to rehydrate or soothe the skin by applying the product for penetration with ultrasound.

F334

Caution:

- Do not use this type of treatment on clients with the following health problems: heart disease, infectious disease, skin infection, skin allergies, clients with pacemakers or metal implants in their body. Note: All jewellery must be removed before performing this treatment.
- Do not allow the probe to be "on" for long periods of time without being used, as "overheating" may occur and damage the probe blade. 3. Do not rub the probe blade or use any abrasive device (sandpaper, metal wool, etc.) as this will damage the blade and possibly cause abrasions to the client's skin. Do not use the probe on any surface other than dermal tissue.
- Never allow the skin surface to dry out during treatment, add more product as needed.
- Take care not to damage the probe cable during use, this can be achieved by not stretching the cable too much, or by ensuring that the cable is not kinked.
- During use, ensure that there is no excess product in the machine or main tower unit to avoid accidental damage. Never use corrosive substances with the ultrasonic peeling probe.
- If the unit does not operate as expected, or if you experience unusual operating circumstances, contact your sales agent immediately for further advice. 7. Never attempt to open the machine or its accessories for inspection, as they contain no "user serviceable parts". Inspection or repair by unauthorised persons is prohibited and will invalidate the manufacturer's warranty.
- If the power cord of the machine is damaged, it must be replaced with a "genuine" part from the manufacturer. Never operate machines with a damaged power cord. Contact your sales agent for more information.

F334

Attention:

- The system operates with a positive ground circuit, for safety reasons. If the handpiece cannot detect the customer's skin (find a return circuit to the machine), the system will stop output, which will be indicated by a "flashing" spatula symbol on the LCD display. This may be caused by a bad connection, or due to a lack of conductive product on the customer's skin. Check that there are no loose connections, that sufficient product has been applied and restart (turn off and on) the machine. Once a good circuit has been re-established, the machine will restart and continue until the selected time has elapsed.
- If the above checks do not rectify the problem when the machine restarts, and the spatula touches the skin, there may be possible damage to the handpiece blade, or damage to the ultrasonic generators or their filaments. The ultrasonic generators and connecting filaments are sensitive parts of the equipment and should be treated as such. Damage may occur if the handpiece is dropped or hit with sufficient force, and is not "user serviceable" and, if you suspect damage, you should contact your sales agent immediately and seek further advice. Use only "original" replacement parts from your dealer. Failure to do so may damage the equipment or cause injury to the user. It will also invalidate the manufacturer's warranty.
- During ultrasonic peeling, the use of "essential oils" or "oil-based" products is prohibited. The skin must be kept moist at all times to avoid overheating of the skin surface. The best way is to keep the handpiece always moving while in contact with the skin. Never use high viscosity skin care products (especially masks) with any ultrasonic peel, as some penetration of the product will occur.
- Before carrying out an ultrasonic peeling treatment, it is advisable to "preset" the power (energy) level of the probe before use. This is so that the level is not too high and causes discomfort to the client, and to prevent the product from spreading to untreated areas.

Contraindications

- Pregnancy and breastfeeding.
- Very sensitive skin.
- Rosacea.
- Seborrheic dermatitis.
- Recent lesions, skin with wounds, infections, psoriasis, dermatitis or unhealthy skin.
- Shaved or depilated skin.
- Skin that has recently undergone deep chemical peeling, burns or surgery.
- Alterations in skin sensitivity.
- Poor healing.
- Previous history of keloids.
- Areas with tattoos on the face or permanent make-up.
- Skin conditions.
- Metal implants.
- Infectious processes or fungus.
- Severe acne.
- Caution when working on tissues with injected fillers, and in the case of clients who have recently had hyaluronic acid, Botox or any other type of product injected, they should wait a reasonable amount of time before having treatment.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com