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Microdermabrasion with Microcrystals

Microdermabrasion equipment using powdered aluminium oxide microcrystals. Ideal for controlled, precise and progressive exfoliation, improves skin quality, reduces blemishes and smoothes expression lines.

Indicated in treatments to eliminate blackheads, acne, accumulated dirt, also to revitalise, nourish, increase the elasticity and oxygenation of the skin.

This non-surgical procedure is performed at the level of the corneal layer and seeks the renewal of the epidermis with mechanical exfoliation, helping to eliminate dead skin by sanding and sweeping, producing cell regeneration, increasing collagen production, and helping to improve the texture and colour of the skin, making it smoother and softer.

Microdermabrasion with microcrystals (aluminium oxide) powder is a very effective, painless and precise mechanical exfoliation method, in which the handpiece spreads the microcrystals on the skin and at the same time causes erosion and a vacuum that drags the dead skin cells and product residues, which are absorbed into the deposit that receives the remains and disposes of them in a very hygienic way. In this case, the impact of these particles against the tissue tends to achieve greater depth in fewer passes.

It can be performed on the face, neck, décolleté, arms and hands, in order to encourage the formation of new cells to replace those eliminated. At the end of the microdermabrasion you may feel a burning sensation and dryness of the skin in the area that was exposed to the procedure, this is normal, to reduce the redness it is advisable to use the cool hammer. However, depending on the intensity of the procedure, the skin may become a redder colour which should disappear in the following hours, but if the discomfort continues, it is preferable to consult a doctor.

To activate the microdermabrasion, select Function Microcrystal and adjust the suction intensity in energy. Place the product in the container.

The skin should be clean and dry. Avoid sun exposure for at least 1 week before. Refrain from using products containing retinoids or glycolic acid for 10 days beforehand.

Microdermabrasion with Microcrystals

It is easy to use and noticeably improves the appearance of the skin, this equipment sprays fine crystals quickly and precisely, on the surface of your skin to produce exfoliation, while dead skin is removed by vacuum suction. The skin will be refreshed and improved after the treatment, it will also stimulate circulation and improve metabolism. Upward, gentle, sweeping movements should be made.

Benefits:

1. Improves the appearance of aging skin, acne scars, and removes dirt from pores.
2. Renews dull and dull skin.
3. Improves facial edema and inflammation.
4. Improves skin elasticity, fine wrinkles, texture, giving it a youthful and fresh appearance.
5. Improve uneven skin tone, pigmentation and other problems.

How to use:

1. Install the equipment, make sure the main body and accessories are well connected, as indicated in the manual. The LCD screen lights up and displays "Microcrystal". The equipment will be in standby mode.
2. The default time is 15min.
3. Press the "+ / -" buttons on the timer to select a working time range.
4. Press the "+ / -" button of "ENERGY", to adjust the suction power.
5. Press the "START/PAUSE" button to start working, while the LCD display shows the working status.
6. When the preset timer has run out, you will hear a "beep" sound. The timer will reset after 15 minutes.

Microdermabrasion with Microcrystals

Precautions for use:

1. Analyse the client's skin type prior to treatment, and select the appropriate strength according to needs.
2. Thoroughly cleanse and dry the skin, no product application is required.
3. After the treatment, cleanse the facial skin and apply a moisturising or nourishing product.
4. The client should avoid exposure to the sun.
5. It is recommended not to use make-up 4 hours after the treatment.
6. Wash the face with cold water, and avoid exposure to the sauna.
7. A certain time interval should be allowed for a new treatment.
8. After 2-3 days the client may experience itching and flaking, so skin protection and care is essential.
9. After exfoliation, please do not use acid type products.
10. The size of the applied crystal should be 180-200mesh. It is disposable and should not be recycled.
11. After treatment, use water and a brush to clean the accessories, then immerse them completely in the container for disinfection using 75% alcohol solution for 15 minutes. Afterwards, remove, clean and place them in the UV steriliser for next use.
12. It is important to cover the client's eyes during treatment on the face to prevent the crystals from entering the eyes.

Contraindications

- Pregnancy and breastfeeding.
- Very sensitive skin.
- Rosacea.
- Seborrhoeic dermatitis.
- Recent lesions, skin with wounds, infections, psoriasis, dermatitis or unhealthy skin.
- Shaved or depilated skin.
- Skin that has recently undergone deep chemical peeling, burns or surgery.
- Alterations in skin sensitivity, unable to differentiate between cold and heat.
- Poor healing.
- Previous history of keloids.
- Areas with tattoos on the face or permanent make-up.
- Skin conditions.
- Infectious processes or fungus.
- Severe acne.
- Caution when working on tissues with injected fillers, and in the case of clients who have recently had hyaluronic acid, Botox or any other type of product injected, they should wait a prudent amount of time before having Hydrafacial treatment.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com