Microdermoabrasión

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Microdermabrasion

Microdermabrasion equipment, using diamond tips. Ideal for controlled, precise and progressive exfoliation to improve skin quality, reduce blemishes, smooth expression lines and treat scars.

Indicated in treatments to eliminate blackheads, acne, accumulated dirt on the skin. Also to revitalise, nourish, increase the elasticity and oxygenation of the skin. Microdermabrasion with diamond tip, is a non-surgical procedure that is performed at the level of the corneal layer and seeks the renewal of the epidermis with mechanical exfoliation, helping to remove dead skin by sanding and sweeping, producing cell regeneration, increasing collagen production, and helping to improve the texture and colour of the skin making it more lush and smooth.

It is a controlled, non-invasive mechanical exfoliation treatment that removes dead skin cells. It improves the appearance, tone, radiance and restores elasticity. Different diamond tips (different steel crowns) with different diameters and granulations can be placed on the handpiece, to be chosen according to the area to be treated. This slides and rubs over the skin, removing the layers of dead skin and the vacuum generated by the equipment in turn sucks up the dead cells.

F336B

It can be performed on the face, neck, décolleté, arms and hands, in order to encourage the formation of new cells to replace those eliminated. At the end of the microdermabrasion you may feel a burning sensation and dryness of the skin in the area that was exposed to the procedure, this is normal, to reduce the redness it is advisable to use the cool hammer. However, depending on the intensity of the procedure, the skin may become redder, which should disappear in the following hours, but if the discomfort continues, it is preferable to consult your doctor.

To activate the device, set the suction intensity under energy.

The skin should be clean and dry. Sun exposure should be avoided for at least 1 week beforehand.

Refrain from using products containing retinoids or glycolic acid for 10 days beforehand.

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Microdermabrasion

How to use:

- Connect the tube to the machine and to the microdermabrasion handpiece.
- Press the "POWER" button.
- Place a foam filter before threading the diamond tip. Remove the filter after each treatment.
- Check that the metal filter is in its proper position on the handle.
- Select the tip you wish to use, there are 9 types of heads, then screw it onto the tip of the handpiece, until the rubber O-ring is tightened.
- Adjust the suction intensity with the vacuum regulator according to the following table, maintaining communication with the client for safety.
- Bring the handpiece with the appropriate diamond tip into contact with the skin. Proceed with a slow and smooth gliding motion. Do not leave the handpiece unmoved for too long on the same skin surface, as this may cause injury.
- Gently stretch the skin with one hand while gliding the diamond tip 2 or 3 times over the same area. Respect this point to avoid redness or injury to the skin.
- You can adjust the suction effect by covering or uncovering the hole on the side of the handle with your finger.
- After the microdermabrasion treatment, cleanse the skin of impurities and dead skin with a soft brush, damp sponge or damp towel.
- Use a brush or cotton swab lightly moistened with alcohol or a disinfectant to clean and disinfect the diamond tip. It is also recommended to sterilise the tips used after each treatment in an ultraviolet steriliser or at high temperature.

Microdermabrasion

Precautions:

- Avoid use on irritated or injured skin areas.
- When decreased suction is noticed, replace the filter.
- Do not use coarse-grained tips on delicate areas.
- Clean the tips used on each person.
- Avoid bumping and dropping the tips, as this can cause damage and loss of polishing power.
- The microdermabrasion treatment cannot be applied on sensitive areas such as the eyelids, and the eyes must be closed during the treatment, it is advisable to cover them with wet cotton pads.
- It is contraindicated in the following cases: skin tumours, moles, birthmarks, angiomas, eyes, herpes and warts.
- Do not attempt to repair the equipment. In case of failure, contact your dealer.
- Do not leave the handle on the same part of the skin for a long time, as this may cause injury.
- The following areas of the skin are not suitable for these treatments: Skin tumours, moles, birthmarks, angiomas, lips, eyes, herpes and warts.
- Do not run the suction pump for prolonged periods without cooling time between treatments, in order to avoid overheating of the pump and damage to the machine.

Indications

- Combat uneven texture and wrinkles.
- Deep facial cleansing.
- Collagen regeneration.
- Dehydration.
- Improve the signs of ageing.
- Acne.
- Improve skin hydration and eliminate the appearance of fine lines.
- Hyperpigmentation and reduction of spots, sun spots.

Care

- The use of sunscreen is recommended after each session.
- Drink 1.5 to 2 litres of water a day to keep the skin hydrated.
- Maintain daily skin care habits, cleanliness and use of creams suitable for the type of skin.
- Facial skin, being thinner, tends to be very delicate and at greater risk of burns, so at no time should any of the handpieces be left in place, and microdermabrasion should be carried out gently.
- In some cases the treatment may cause reddening of the skin, irritation and swelling, which will improve with time.
- It is important to clean the handpieces properly after each procedure.

Warnings

- This treatment is not risk-free, which is why it must be carried out by a professional aesthetic specialist.
- Follow at all times the recommendations of the professionals of the aesthetic centre.
- Never, under any circumstances, attempt to open or inspect the internal components or accessories of the machine. If inspection is necessary, please contact your supplier or a qualified technician.
- Never use the unit in damp or insufficiently ventilated environments, such as water areas, bathrooms, etc., in order to prevent damage to the electrical components of your equipment.
- Never use the equipment outdoors, inclement weather may cause damage.
- If the electrical cable is broken, ask a professional to supply you with another one and replace it, in order to avoid risks, or contact the supplier.
- At the end of the session, it is important to clean the handpieces with an antiseptic product after each treatment.
- Check with the client throughout the treatment that the intensity is appropriate.
- It is recommended that the client drinks plenty of water before and after the treatment.
- Do not place any liquid on the equipment.
- Be careful with electrical connections.
- Please disassemble and assemble the equipment by professional personnel.
- In case of malfunction of the device, check all connections carefully and contact a service technician in case of doubt.

Contraindications

- Pregnancy and breastfeeding.
- Very sensitive skin.
- Rosacea.
- Seborrhoeic dermatitis.
- Recent lesions, skin with wounds, infections, psoriasis, dermatitis or unhealthy skin.
- Shaved or depilated skin.
- Skin that has recently undergone deep chemical peeling, burns or surgery.
- Alterations in skin sensitivity, unable to differentiate between cold and heat.
- Poor healing.
- Previous history of keloids.
- Areas with tattoos on the face or permanent make-up.
- Skin conditions.
- Infectious processes or fungus.
- Severe acne.
- Caution when working on tissues with injected fillers, and in the case of clients who have recently had hyaluronic acid, Botox or any other type of product injected, you should wait a reasonable amount of time before having treatment.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com