F829

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F829

Our F829 is an effective equipment to reduce spots, moles, age spots and sun damage, freckles, spider veins and small superficial wrinkles.

It works on the basis of a high voltage discharge, but very low current, which allows the spot to be transformed into gas by plasma ionisation. The micro blood vessels will quickly solidify without bleeding, and a carbon crust will form, protecting the skin from infection.

The device acts by localised epidermal ablation, removing damaged skin and promoting skin rejuvenation. The treated area will be swollen for the first two days, followed by a small scab and slight redness for approximately seven days.

The spot removal has various intensity levels for different treatments. For mild spots a low level of intensity will be used, and the high level is for moles and stronger spots. To regulate the intensity, we will always go from low to high, increasing it if necessary.

It is extremely important before using the equipment to clean the area where we are going to work and a thorough evaluation of the client, verifying that there are no melanomas on the skin, it is always recommended to consult with the treating physician.

How to use:

- Connect the handpiece cable to the equipment and attach the appropriate tip. The type of tip to use is at the discretion of the practitioner. In general, reusable tips are used for normal or large stains, while disposable tips are indicated for removing small or very fine stains.
- Once selected, screw the coarse needle into the tip of the handpiece. This is reusable, and is used to treat large areas, it must be sterilised after each use.
- To use the thin disposable needles, unscrew the thicker needle from the handpiece, then take out the thin needle, put it inside the handpiece and screw the metal head with a hole on top of it, they are used for small spots where more precision and detail is required.
- Turn the equipment on by pressing the "ON" button.
- Once the equipment has been switched on, the power level can be adjusted on the knob, increasing the intensity to the desired level using the "R/ENERGY" rotary knob.
- Hold the knob like a pen.
- To start working, sanitise the working area and start making contact with the tip of the handpiece on the area of the skin with the stain to be treated.
- Bring the tip of the handpiece close to the spot to be removed until a spark is produced between the tip and the spot, and quickly move the handpiece away from the skin. The tip should not touch the skin.
- When finished, turn down the intensity to minimum using the "R/ENERGY" rotary knob and press the "OFF" button to switch off the device.
- Unscrew the tips. Clean and sterilise the reusable tip and metal head and discard the disposable tips.
- This equipment is designed for stain removal. In any case, it is best to consult a dermatologist.
- If the spots are not completely removed, repeat the previous steps. If they are very large, it is best to remove them in several sessions.

Precautions:

- Because of the varying thickness of the skin, spot removal results may vary. Therefore, it is advised to do a skin test prior to treatment to get the best possible results.
- Do not apply this treatment around the eyes.
- It is not recommended to apply the treatment on veins.
- After removal of the spot the area will become reddened, this is a normal reaction and will gradually disappear.
- After a few days a scab or scar will appear, do not remove it with fingers or nails to avoid infection.
- After treatment, apply a suitable repair product to promote skin regeneration after removal of the blemish.
- Be sure to lower the intensity to a minimum after use to avoid electric shocks or burns on the next patient.

Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Cami de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com