

F315A



The epidermis is made up of the keratinous and granular stratum. These squamous cells develop and die continuously along with the metabolism of the skin, turning into dead cells that generate a scaly layer and prevent the oxygenation of the skin, giving it a dull appearance. The rotating brushes remove these dead skin cells or accumulated dust in a completely mechanical way. With this treatment we accelerate the vitality of the skin and keep it healthy.

The brush works as follows: it is passed over the entire face in circular movements. You can divide the face by zones and dedicate 10 seconds to each one, working each zone separately: cheeks, nose, chin and forehead.

The facial cleansing kit comes with 1 rotating handpiece, 4 different heads, 2 face brushes (1 small and 1 large), 2 stones, small and large. Suitable for a complete cleansing of the skin, where the aim is to remove dirt, in a gentle way, with the brushes, or deeper and more abrasive, with the stone, achieving a brighter skin immediately.

It is a tool that facilitates the cleaning of make-up, dirt, grease and the removal of dead skin. It also improves the texture and imperfections, clogged pores, blackheads and pimples, as well as effectively reaching the most difficult areas to cleanse, such as the sides of the nose and the T-zone. Through a pleasant massage, the blood circulation is also stimulated.

It is recommended to use a soapy product, facial cleanser or exfoliator, the brush should be moistened before starting the treatment. To use it, turn it on and gradually increase the speed to "ENERGY", depending on tolerance and the type of cleansing, use a low speed for sensitive skin and a high speed for normal skin.

F315A



It is important to clean and disinfect the brushes with warm water and a little soap after each treatment.

How to use:

Plug the brush handpiece connector into the rear socket. Choose the most appropriate brush head according to the client's skin type. Generally, dry or delicate skin requires the brush head with the thickest bristles. Stiffer bristle brushes are suitable for oily and stubborn skin.

Turn the power on and set the "rotary brush" switch to ON. The appliance goes into stand-by mode.

Set the "R/Energy" control. The brush will start rotating. The rotation speed will increase with increasing intensity. For operation, it is important to select an appropriate rotation speed according to the sensitivity of the client's skin.

Once the treatment is finished, soak the brush head in clean water with some cleaning product for 5 minutes. Once clean, soak it in alcohol for half an hour.

The brush head should be disinfected with ultraviolet radiation and stored dry in a completely sterile container.

Precautions:

When using the equipment, it is forbidden to exert too strong pressure with the brush on the skin.

Do not use the device with alkaline or acidic substances.

Once the treatment is finished, switch off and unplug the unit.

Do not use this equipment if the skin is sensitive or if the capillaries are in poor condition.

The brushes should be cleaned and sterilised each time the unit is used.



Yubitzay Avila
Customer Training
T. +34 93 824 9020
y.avila@weelko.com
S. Yubitzay Avila

Weelko Barcelona S. L.
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 9020
info@weelko.com
www.weelko.com