MultiEquipment. Ref F-333A

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

Radiofrecuencia
Radiofrequency
Radiofréquence
Radiofrequenzgerät
Radiofrequenza

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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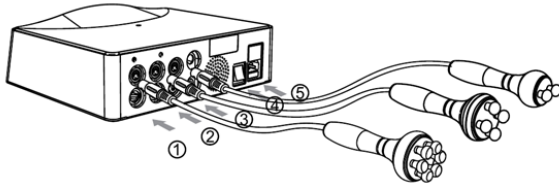
Instructions of use

1. Connect all the accessories as shown in the illustration above.
2. Press the power switch ON/OFF, number 4 on the picture, and the LCD screen will be on.

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3. Select the desired function and press ENERGY button to adjust the energy level, then press TIME button to set the operation timer. Press STATE button to choose the wave type as needed.
4. Clean the treatment area.
5. Apply specific gel or serum to body parts to keep skin hydrated prior to treatment. Please ensure that the electrode is in touch with the skin of the patient as much as possible to avoid the accumulation of currents which will cause skin irritation or redness. Add half bottle (1 ml) or wrinkle removal serum on the face, including the eye area.

6. Set timer to 15 min.

Treatment plans:

- A. 30 min. for facial treatment. If this is the first treatment for the patient, 15 min. for each side of the face is suggested.
 - B. 10 min. for the eyes.
 - C. 30 min. for the breast and other parts of the body.
- The total treatment time for the whole body should not exceed 1h30 each time.
7. Choose the appropriate probe size, energy level and output waves.
 8. Put 2 ml of high resistance wrinkle removal cream on top, bottom, left and right side of the face without spreading the cream, or choose to put the cream on, before the treatment of the particular area.
 9. After adjusting the above settings, press START/PAUSE button to begin the treatment.
 10. Once you release the foot controller, the electrode will stop working.

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Cautions

- Radiofrequency treatments must not extend over a long period of time. The recommended duration is 10 minutes for facial treatments and 20 minutes for body treatments. In any case, treatments extend beyond 60 minutes.
- To avoid burning to the customer, never forget to put this plate in contact with the skin.
- Apply the electrodes on the customer's skin making constantly circles, without stopping of making movements. Not keep the electrode without moving at the same zone during a long period of time, the customer could be burned.
- Don't forget to apply gel or cream in the treatment zone.

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Secondary effects and contraindications

The secondary effects of the radiofrequency are:

- Redness and swelling after the session (are transient, but usually present).
- Some small ecchymosis (rare).

The radiofrequency treatment is contraindicated in:

- Pregnancy and lactation.
- Metal prostheses or pacemakers.
- Serious heart defects.
- Coagulation disorders.
- Connective tissue diseases and neuromuscular.
- Cancer.
- Patients with overweight.
- Recent collagen implants.

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